



# Synergy Vision™ Video Integration Guide

## Synergy Vision Connect™ Live Case Video Routing

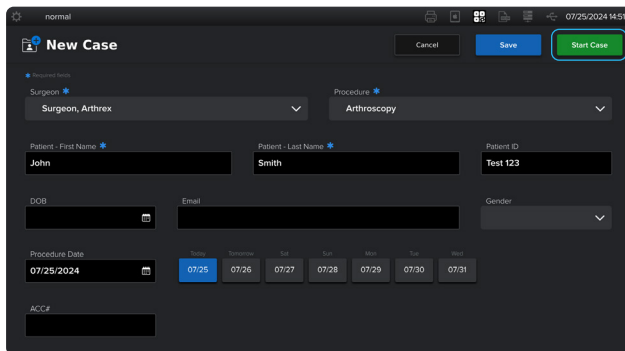


01

Enter the patient's information, and then press Start Case.

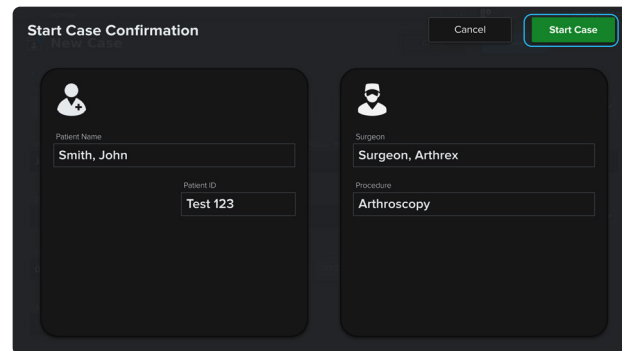


02

Once the case is confirmed, press Start Case again to begin.

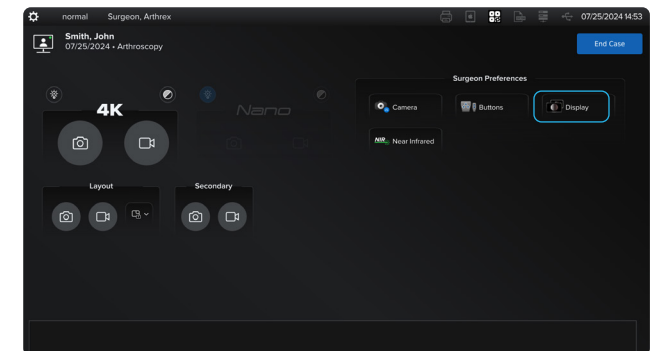

03


Press the Display icon to access the video routing screen.

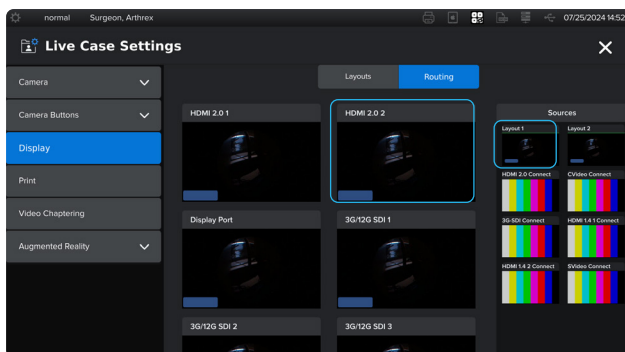

04


Touch the source to be routed. A green frame around the source will indicate that it has been selected for routing. Touch the desired display again to route the source.

## Synergy Vision Connect™ Out-of-Case Video Routing

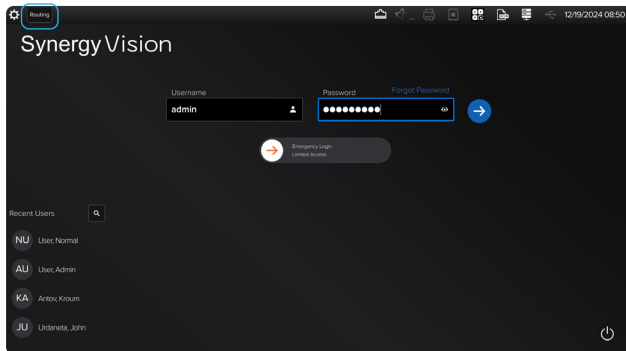


01

Tap the Routing icon on the top-left corner to enter the out-of-case routing screen.

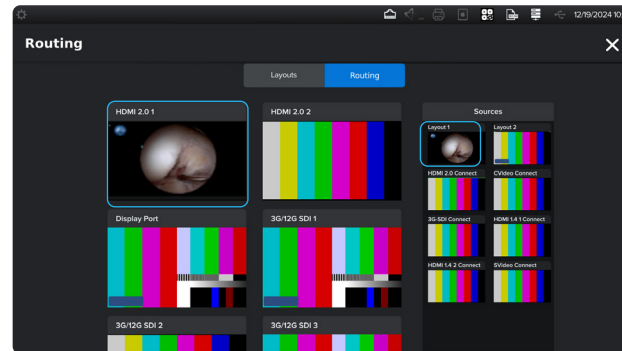


02

Touch the source to be routed. A green frame around the source will indicate that it has been selected for routing. Touch the desired display again to route the source.

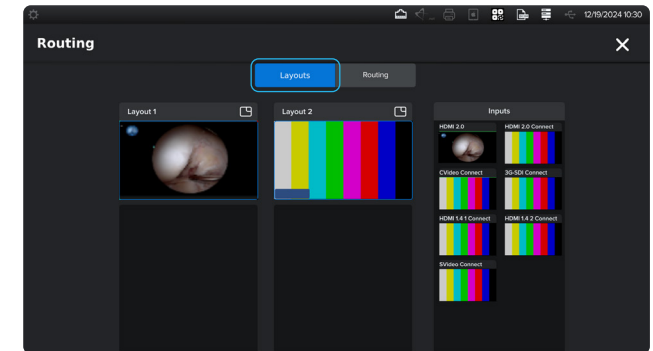


03

The layout options for out-of-case routing can be accessed by selecting the Layouts tab.

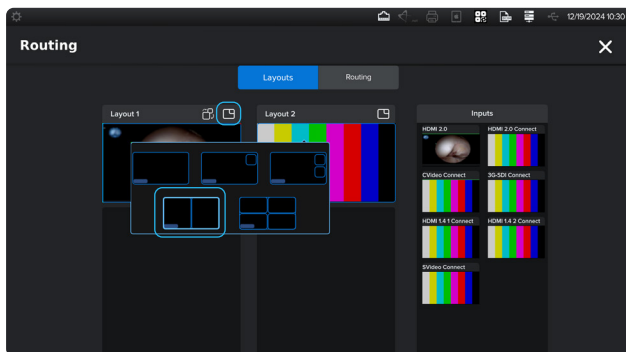


04

To choose a layout configuration, tap the layout window icon at the top-right corner of Layout 1 or 2.

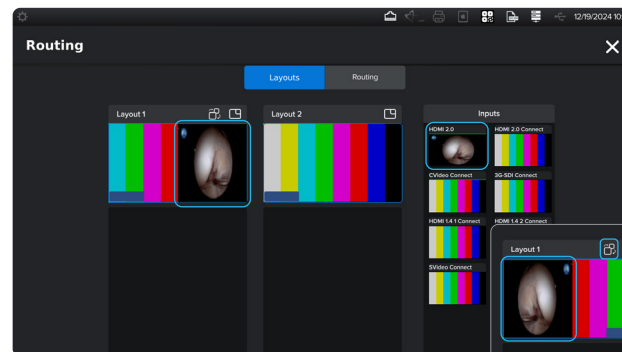

05


Choose a source to populate the layout configuration. Optional: In side-by-side layout, you can flip video sources within the layout configuration by selecting the flip image icon.

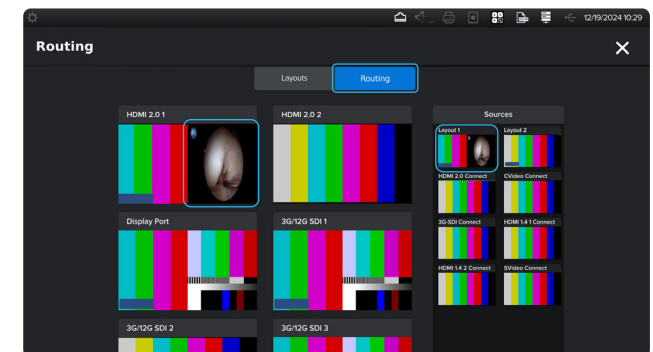

06


Return to the Routing tab. Tap the Layout 1 or 2 source to be routed. A green frame around the source will indicate that it has been selected for routing. Touch the desired display again to route the layout source.

## Synergy Vision Connect™ Layout Configuration

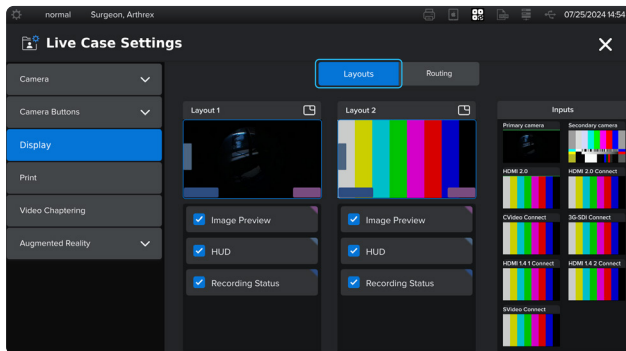


01

The layout options can be accessed by selecting the Layouts tab.

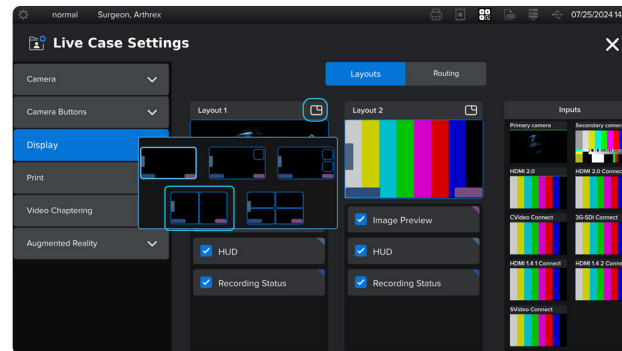


02

To choose a layout configuration, tap the layout window icon at the top-right corner of Layout 1 or 2.

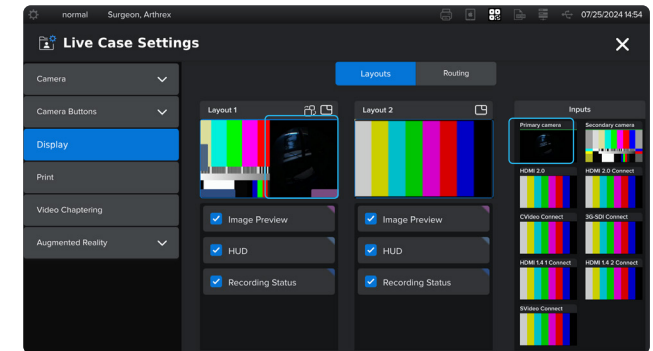


03

Choose a source to populate the layout configuration.

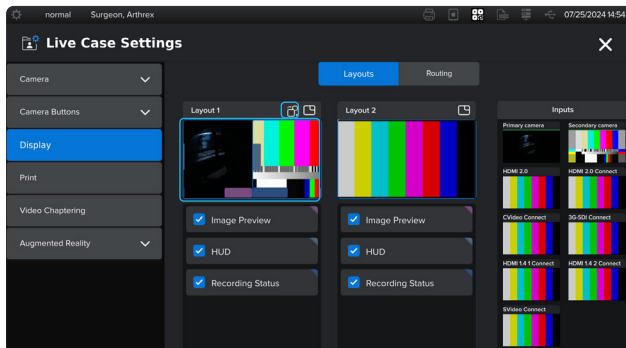

04


Optional: In side-by-side layout, you can flip video sources within the layout configuration by selecting the flip image icon.

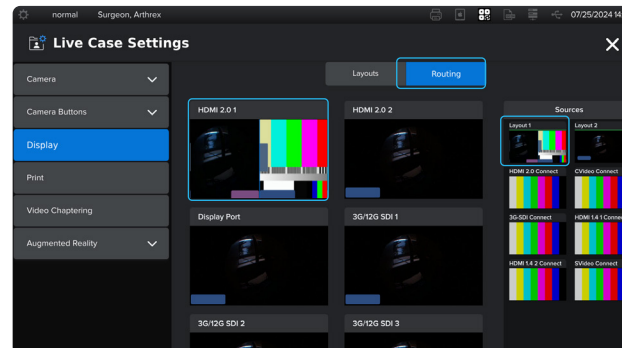

05


Return to the Routing tab. Touch the Layout 1 or 2 source to be routed. A green frame around the source will indicate that it has been selected for routing. Touch the desired display again to route the layout source.

## Synergy Vision Connect™ Administrator Naming Sources and Destinations

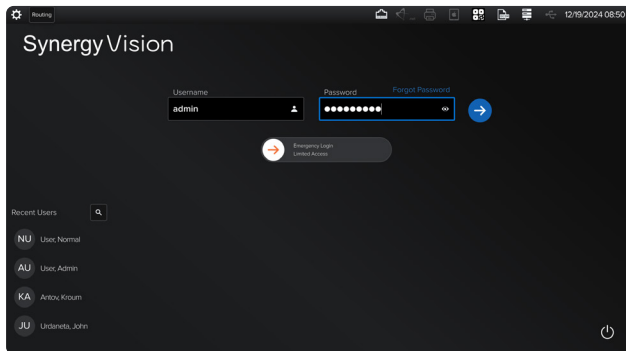


01

Login as the admin user to change the source and destination names.

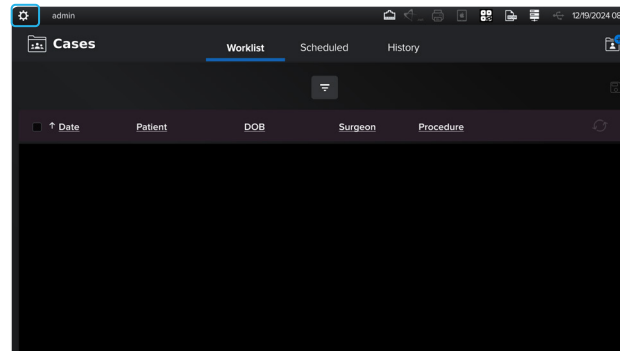


02

Access settings by tapping the gear icon.

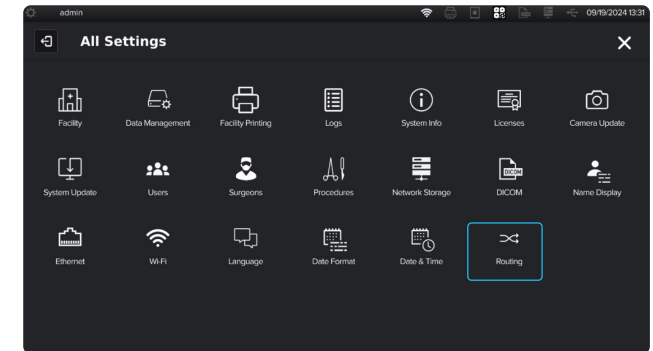


03

Tap the Routing icon to enter the destination and source settings.

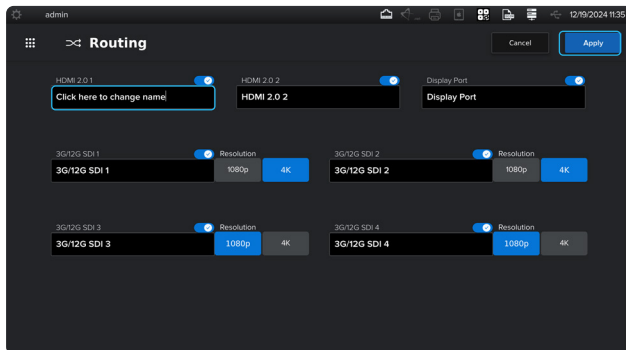

04


Tap on a destination to change the name, and then tap Apply.

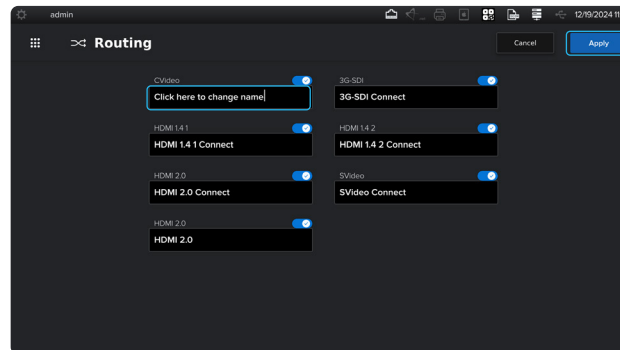

05


Tap on a source to change the name, and then tap Apply.

## Synergy Vision Connect™ Administrator Enabling/Disabling Sources and Destinations

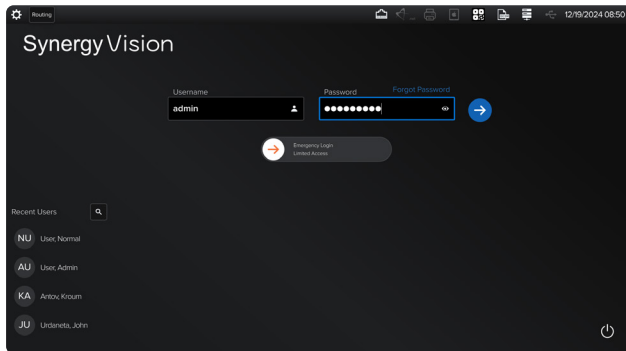


01

To enable and disable sources and destinations, log in as the admin user.

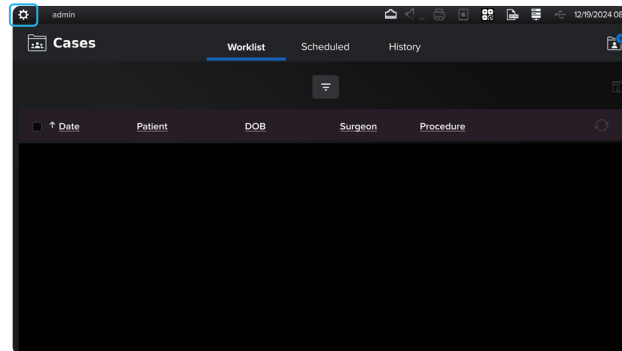


02

Access settings by tapping the gear icon.

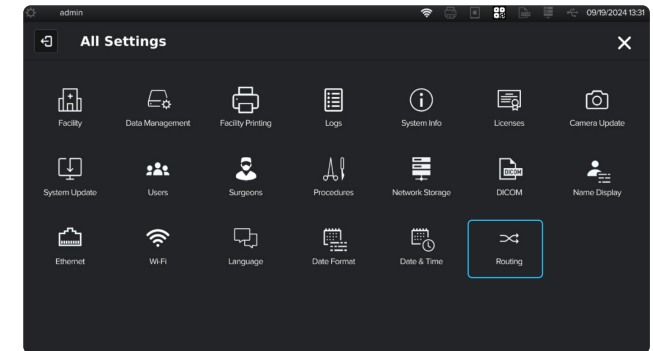


03

Tap the Routing icon to enter the destination and source settings.

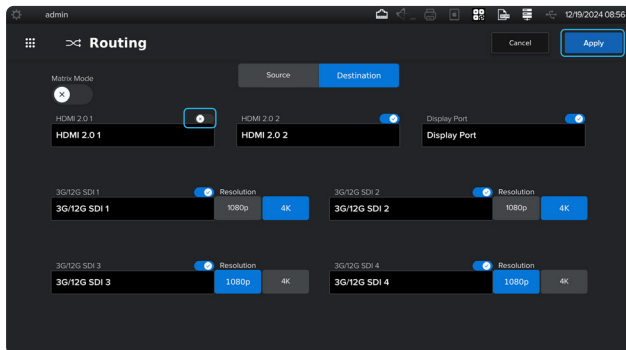

04

Select the slider button to enable or disable a destination, and then tap Apply.

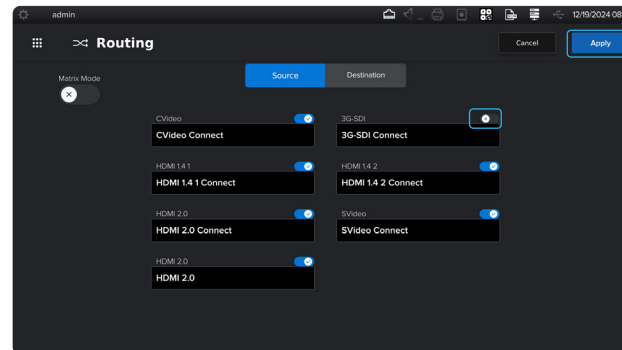

05

Select the toggle switch to enable or disable a source, and then tap Apply.